**Occlusivfolien - Occlusion Foils - Feuilles d'Occlusion - Fogli Occlusivi
Lamina Oclusión - Película Oclusão - Okklusion-Foliet
Okklusjonsfolien - Occlusiv-Folien - Okklusiokalvo - Occlusie Folie**

Occlusivfolien

DE

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EN

Product Description:

The occlusion foils are thin foils that may be applied to the back surface of one lens of a patient's glasses. It is intended to use them for treatment and therapy of amblyopia for children and adults.

Indications:

The occlusion foils are thin ophthalmic foils intended to be applied to spectacle lenses to provide an optical effect.

Contraindications:

None

Warnings:

The occlusion foils are not intended for the creation of impact resistant safety eyewear.

Precautions:

- Patients should be informed that the occlusion foils will modify their spectacles. Their vision may be altered particularly in unusual lighting conditions.
- Avoid exposing the occlusion foils to dust, dirt, chemical solvents, grease or oils.

Fitting:

1. Cut the foil to the shape of the lens, 1mm smaller.
2. Clean and degrease the lens and the shiny side of the foil.
3. Place the foil on the wet inner side of the lens, press on it and let it dry until you can see no air bubbles.
4. Leave for at least one hour to achieve the designed degree of occlusion.

Advice:

Please report a serious incident occurring in relation to these products to Ryser Optik AG and the local competent authority (EU) or local regulatory authority.

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Dieter Mann GmbH