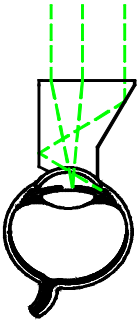


Ocular Mori Upright Surgical Gonio Lens						
	Product Code	Gonio Image Mag	Contact OD	Lens Height	Static Gonio FOV	 Developed With: Kazuhiko Mori MD, PhD Kyoto Japan. Journal Reference: American Journal Of Ophthalmology, Vol 143, No. 1, pp 154- 155, January 2007.
	OMUSG	0.80x	11.5	21.6mm	110°	
	CE					

Design

- The Mori Direct View Gonio Lens is designed for glaucoma procedures, including goniosynechialysis, trabeculotomy goniotomy and trabecular stent placement.
- The two mirror design redirects the oblique Gonio image to the coaxial “cataract” surgical position, allowing the surgeon easy 360° viewing of the anterior chamber.
- Central view to observe instruments passing across anterior chamber.
- Combines the most favorable features of traditional gonio prisms while providing a properly orientated view of the angle.
- It is lightweight and directly hand held which lends itself naturally to delicate maneuvers while observing the anterior chamber angle.
- Anterior chamber charting made easier with correct image orientation.
The mirrored surfaces are silvered with an exclusive double layer, protective coating to prevent peeling and damage under normal daily use.
- Works best with coaxial light source.

Cleaning

Rinse: Immediately upon removal from patient's eye, thoroughly rinse in cool or tepid water.
Wash: Place a few drops of mild soap on a moistened cotton ball. Gently clean with a circular motion.
Rinse: Thoroughly rinse in cool or tepid water, then dry carefully with a *non-linting* tissue.
Then: Proceed with either disinfection or sterilization instructions.

Disinfection

Soak In:	GLUTARALDEHYDE	OR	BLEACH
	2% or 3.4% aqueous solution		10% solution mixed at: 1 part bleach to 9 parts cool tepid water
	Temperature per manufacturer instructions		
	Minimum exposure time = 20 minutes		
	Caution To avoid damage to the lens, do not exceed recommended exposure time.		
Then:	Rinse lens <i>thoroughly</i> to remove disinfection solution. 3 cycles of 1 minute, with cool or tepid water is recommended. Dry carefully and place in a dry storage case.		
NOTE	This lens is known to be compatible with: Asepti-Wipe, Cavi-cide, Cidex, Cidex OPA, DisCide Wipe, Enviro-cide, H ₂ O ₂ - 3%, and Opti-Cide		

Sterilization						
AUTOCLAVE	STERRAD	STERIS SYSTEM 1	ETO	ETO Parameters		
No	No	No	YES	Minimum Time	Temperature	Aeration Time
			See Right	1 hour	130°F (54°C)	12 hours
WARNING	Never Steam Autoclave or Boil listed lenses. Never soak in Alcohol, Acetone or Other Solvents.					

For information on compatibility with alternative product care methods, contact Customer Service.



2255 116th Ave NE, Bellevue, Washington 98004-3039 USA
T: 425-455-5200 or 800-888-6616 F: 425-462-6669

E: ocular@ocular-instruments.com I: www.ocular-instruments.com

© 2001 Ocular Instruments
10098A3017